HR/BIH Priručnik za korisnike

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Verzija: V20211229

Table of contents

1. Safety Information	96
1.1 Important Safety Information	96
1.2 Labels	96
2. Product introduction	96
2.1 Product Name	96
2.2 Trade name	96
2.3 Indications	96
2.4 Intended Use	96
2.5 Purpose of Continuous Glucose Monitoring Systems	96
2.6 Contraindications	96
2.7 Blood Glucose Unit	96
2.8 What the POCtech CGM APP does	97
2.9 How to set-up the POCtech CGM APP?	97
3. Main screen	97
3.1 Interface definition	97
3.2 Main Menu	98
3.3 Sub-Menu	98
3.4 Icons	101
4. Start a new test	102
4.1 Implant the sensor	102
4.2 Start POCtech CGM APP	102
4.3 Prepare the transmitter	102
4.4 Connect the APP with the transmitter	102
4.5 Install the transmitter to the sensor immediately	103
5. Calibration	103
5.1 You need to calibrate the system after initialization	103
5.2 How to calibrate	103
5.3 Subsequent Recalibration	105
5.4 When to calibrate	105
6. Event Inputs	107
7. Features of the Touch-screen display	108
7.1 Blood Glucose Trend Arrow	108
7.2 Blood Glucose Graph Color	108
7.3 Blood Glucose in Hyper / Hypo States and alerts	109
7.4 Left/Right Page Turns	110
7.5 Review Glucose Point Value	110
8. Hypoglycemia and Hyperglycemia alert	110
9. Ending your sensor session	110
9.1 Ending automatically	110
9.2 Force Stop	110

10. Remove the sensor together with the transmitter.....	110
11. Review data.....	110
11.1 The sensor signal	110
11.2 Review & Export Data.....	110
11.3 24-hr Overlay	110
11.4 View Report	112
11.5 Share data	113
12. Settings	113
12.1 Customizing app alerts.....	113
12.2 Setting high/low alerts	114
12.3 Close the alert of audio and vibrator.....	114
13. Maintenance.....	115
13.1 Make sure that your smart phone Bluetooth is on	115
13.2 Manage the app to stay active.....	115
13.3 Carry your smart phone at all times.....	115
13.4 Never clean up the POCtech CGM APP cache	115
13.5 Update software version.....	115
13.6 Troubleshooting.....	115
14. Performance.....	116
14.1 Software Performance.....	116
14.2 Stability and accuracy.....	116
15. Manufacturer information	116

1. Safety Information

1.1 Important Safety Information

Read this entire manual before using your Continuous Glucose Monitoring Systems. If you do not understand some of the content or have any questions, please ask your Diabetes Management Team or health care provider. Contraindications, warnings, precautions, cautions, and other important safety information can be found in this User Manual.

- This device is not designed to replace a blood glucose meter. The system must be used with a blood glucose meter.
- You can't make a treatment decision based on Continuous Glucose Monitoring Systems alone. You should be under the guidance of a doctor or take a finger blood glucose to confirm.
- The Device is mainly used for detecting glucose trends. Symptoms related to low or high blood glucose levels should not be ignored. If you feel uncomfortable, you should carry out medical activities under the guidance of medical institutions or use your blood glucose meter to verify the results.
- You should update the device's calibration according to the instruction in this manual to ensure device performance. The performance of the device has not been studied when calibrated less frequently than the recommendation.
- The CGMS product should be used under the supervision of a physician or users should be trained.
- CT-Scan, X-Ray or MRI: When must be examined with CT, X-ray, or MRI imaging, remove the transmitter for the processes and place it back afterward. The interruption will cause the sensor signal to re-stabilize. Some data loss may occur. An additional BG calibration is required.
- Avoid Extreme Temperature: Try not to enter or stay in extreme temperature that is outside the specified range of the system operation.
- Strong Electromagnetic Fields: Avoid strong electromagnetic fields.

Preparation and installation of Transmitter and sensor: (See the CGMS User Manual for details.)

1.2 Labels

Symbol	Meaning of symbol
	Manufacturer
	Authorized representative in the European Community
	Consult instruction manual

2. Product introduction

2.1 Product Name

Software for Continuous Glucose Monitoring Systems

2.2 Trade name

POCTech CGM APP

2.3 Indications

Software for Continuous Glucose Monitoring Systems is indicated for detecting trends and tracking patterns in adults with diabetes. It is intended for use by patients.

2.4 Intended Use

Software for Continuous Glucose Monitoring Systems is a mobile application used in combination with Continuous Glucose Monitoring Systems manufactured by POCTech Co., Ltd. This software will be used to collect, transform the blood glucose signal collected by the transmitter and display, store, and analyze the glucose value and graph in real time.

2.5 Purpose of Continuous Glucose Monitoring Systems

See the CGMS User Manual for details.

2.6 Contraindications

CT-Scan, X-Ray or MRI: When must be examined with CT, X-ray, or MRI imaging, remove the transmitter for the processes and place it back afterward. The interruption will cause the sensor signal to re-stabilize. Some data loss may occur. An additional BG calibration is required.

2.7 Blood Glucose Unit mmol/L or mg/dL

2.8 What the POCTech CGM APP does

Smartphone that support BLE (Version 4.0 or up) download with POCTech CGM APP can replace the Receiver of Continuous Glucose Monitoring Systems, and is used together with Transmitter and Sensor.

POCTech CGM APP was created to work with your smart phone, providing more options in monitoring your glucose trends and patterns. The App is similar to all other apps.

Note: The App

- ① Works with only smart phones with Bluetooth 4.0 or up.
- ② Connects to a Transmitter in an exclusive mode. The transmitter will not communicate with other Bluetooth devices at the same time.
- ③ Must be used together with products manufactured by POCTech.
- ④ Version update may be necessary from time to time.

2.9 How to set-up the POCTech CGM APP?

Download the POCTech CGM APP file from the app store. Install the app following your smart phone instructions. The POCTech Icon will appear on your smart phone if the installation is successful.

3. Main screen

3.1 Interface definition


3.2 Main Menu

Button	Button
	Sensor Main Menu (default);
	Settings Menu
	User Input Menu
	Data Review/Export Menu
Click Here to Turn Sensor Search/On/Off >>>	Go to Sensor Search / Connect

3.3 Sub-Menu

3.3.1 Settings Menu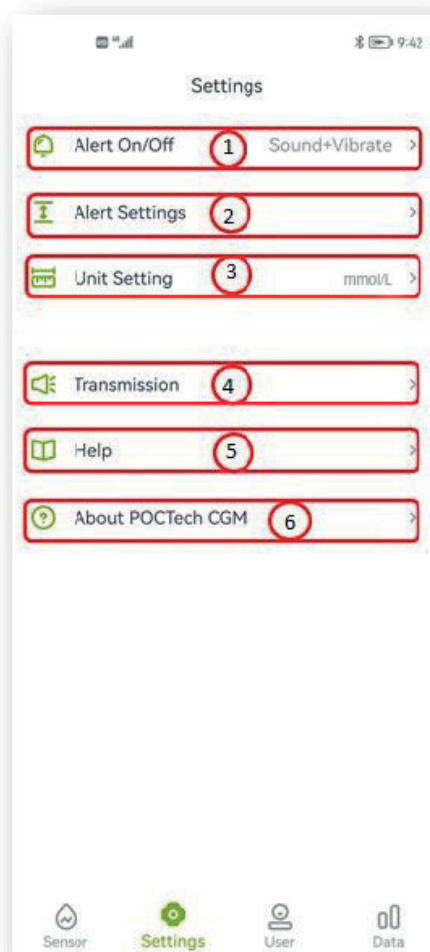


Menu No.	Menu Item	Setting...
①	Alert On/Off	Choose Audio, Vibrate, Sound+ Vibrate or Off
②	Alert setting	Set High Alert level and Low Alert level
③	Unit Setting	Choose mmol/L or mg/dL
④	Transmission	Default Closed. If opened, it will broadcast glucose internally.
⑤	Help	Obtain User Manual.
⑥	About POCtech CGM	Obtain information about Manufacturers, Label, Version, etc.

Note


For more description about the Settings function, please refer to Section 12.

3.3.2 User Input Menu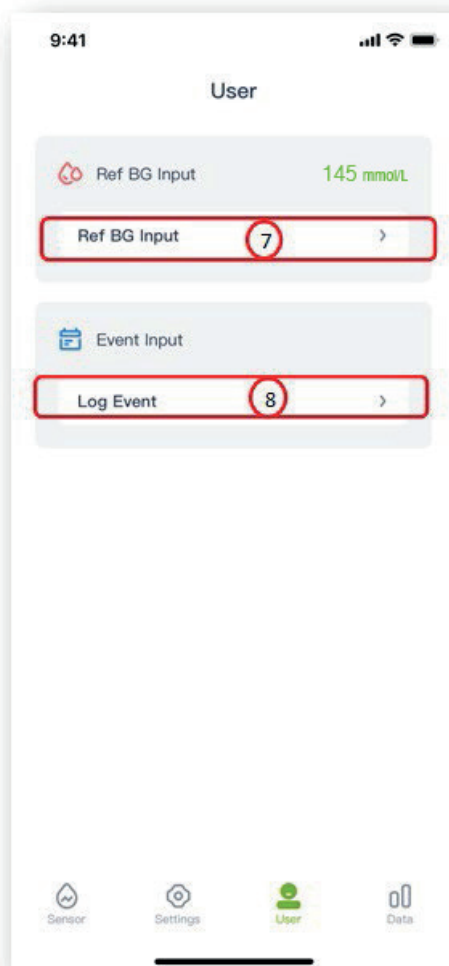

Menu No.	Menu Item	Enter OR Log...
⑦	Ref BG Input	Log finger blood glucose
⑧	Log Event	Log events, such as meal, insulin, medicine or exercise

3.3.3 Data Review/Export Menu

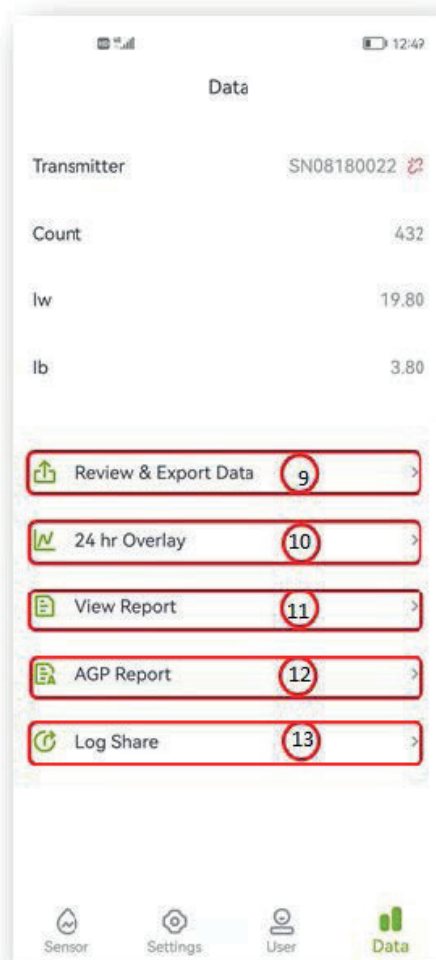Menu No.	Menu Item	Select to View OR Export...
⑨	Review & Export Data	Choose history record and review blood glucose graph
⑩	24 hr Overlay	Choose history record and review blood glucose with 24hrs Overlay graph
⑪	View Report	Choose history record and generate patient reports
⑫	AGP Report	Report compliance with AGP specifications
⑬	Log Share	System logs during the wearing period

Note

More describe about the Review data function, please refer to Section 11.

3.4 Icons

	Alert Indicator: Sound + Vibrator
	Alert Indicator: Sound
	Alert Indicator: Vibrator
	Alert Indicator: No Sound or Vibrator
	Calibration Prompt
	It indicates a status of an abnormal current.
	It represents the state of abnormal current recovery.
	It indicates a status above or below the set limit.
	It indicates an entered blood glucose.
	It indicates a logged meal event.
	It indicates a logged an insulin event.
	It indicates a logged a medicine event.
	It indicates a logged an exercise event.

	Blood glucose is in steady state
	Blood glucose increases moderately
	Blood glucose increases rapidly
	Blood glucose decreases moderately
	Blood glucose decreases rapidly
	Blood glucose decreases rapidly, and is below or near low limit
	Open or On
	Close or Off
	Transmitter is connected.
	Transmitter is disconnected.
	Urgent Low / High Alert
	Low / High Predictive Alert
	Alert - Synchronizing

4. Start a new test

4.1 Implant the sensor

Prepare and implant the sensor. (See the Sensor preparation of the CGMS User Manual for details.)

4.2 Start POCTech CGM APP


Click the Software ICON on the smart phone desk to open the APP.

4.3 Prepare the transmitter


Prepare the transmitter. Install a CR1620 battery. (See the Transmitter preparation of the CGMS User Manual for details.)

4.4 Connect the APP with the transmitter


4.4.1 Click on the line “Click Here to turn sensor Search/On/Off >>>” to go to the sensor search.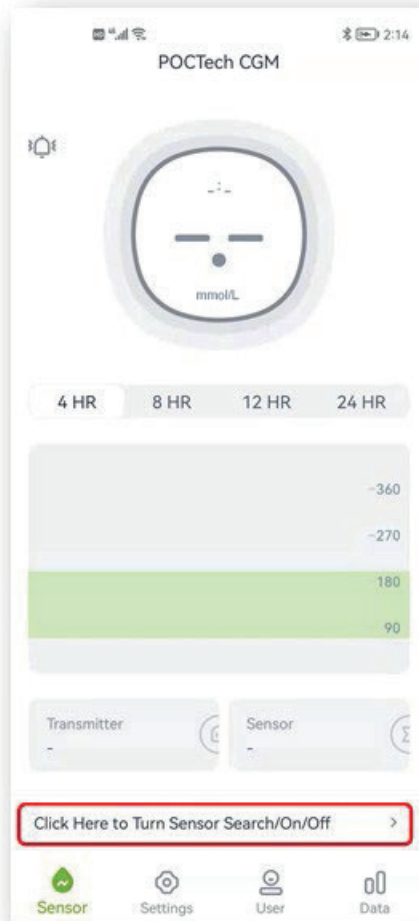


4.4.2 Turn the Phone Bluetooth “ON” if it is “OFF”, then touch “Search” and allow for a few seconds for the App to find the transmitter. The active transmitter serial number will appear on the list;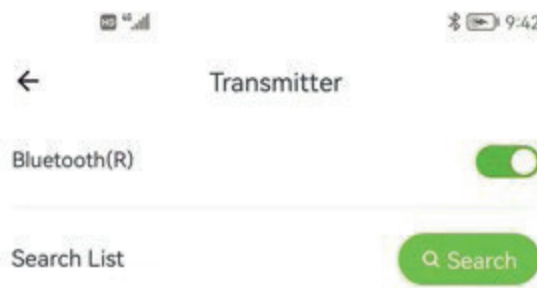

4.4.3 Select the transmitter number (verify on the transmitter id) to connect by clicking the listed transmitter number to pop up dialog box.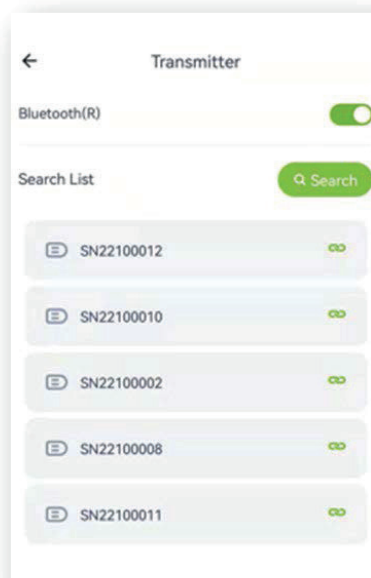

4.4.4 Then choose 'OK', the app will conduct a handshaking and connect with the transmitter. Once locked, an initialization process begins and a window shows a count-down clock.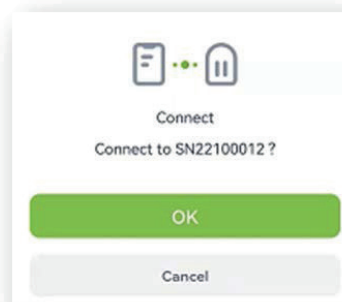


4.5 Install the transmitter to the sensor immediately.
(See the transmitter installation of the CGMS User Manual for details.)

5. Calibration

5.1 You need to calibrate the system after initialization.

5.2 How to calibrate


Step 1: When the initialization period is complete, a pop-up window will be displayed to prompt you to enter finger blood glucose value to calibrate the sensor;

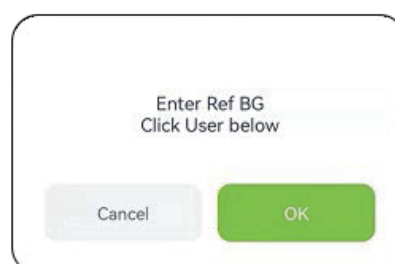Step 2: Take a finger blood glucose measurement, enter the BG value within 5 minutes.

A dialog box titled "Ref BG Input" with the unit "mmol/L" in the top right corner. It contains three input fields with the values 102, 103, and 104. At the bottom, there are two buttons: "Cancel" and "OK".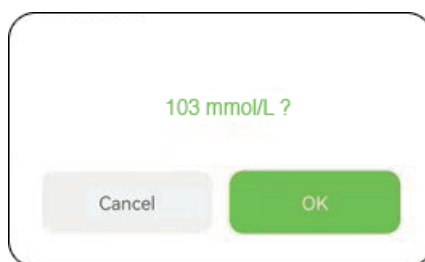

A confirmation dialog box with the text "103 mmol/L ?" in green. At the bottom, there are two buttons: "Cancel" and "OK".

Step 3: Press "OK" to confirm.

Note

When taking a finger blood glucose measurement, it is important to make it as accurately as possible. Make sure you wash and dry your hands.


You can re-check the correctness of your BG input by opening the User Input  menu.

A menu titled "User" with a red circular icon containing a white 'u'. It shows "Ref BG Input" with the value "145 mmol/L". Below this is a dropdown menu with "Ref BG Input" and a right-pointing arrow.

5.3 Subsequent Recalibration

Follow Table 5.4, for recalibrations during the monitoring. Enter finger BG from the User Input '👤' menu or in responding to calibration prompt. (👤)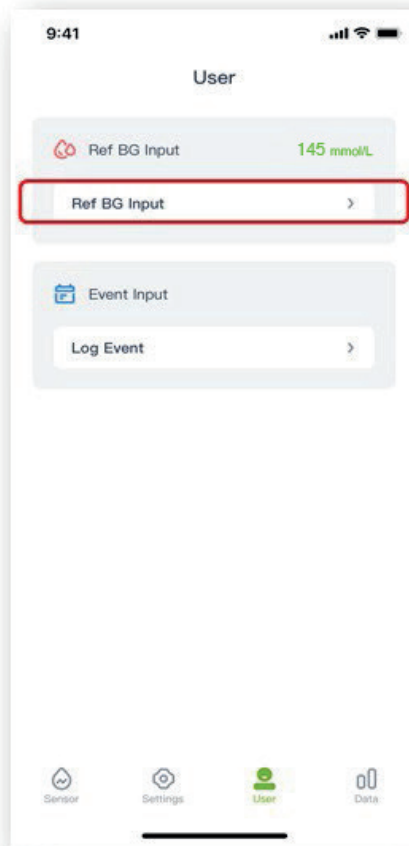

5.4 When to calibrate

Follow the Table below to carry out routine calibrations.

Day	Recalibration Times
Starting Day	Cal 1, after initialization Cal 2, before dinner or before bed
2 nd Day	Cal 1, fasting BG (before breakfast) Cal 2, before dinner
3 rd Day & after	Fasting BG only (once every day)

Note: Add an additional calibration if obvious sensor deviation is suspected.

 **Warning**

Entering the wrong BG values, or waiting more than 5 minutes before entry, might affect sensor accuracy.

 **Warning**


There may be significant differences between finger blood glucose and Subcutaneous fluid glucose during periods of rapid blood glucose changes.

 **Warning**

The performance of this product using less frequent calibrations has not been studied.

Note

- ① Calibration times must be in glucose steady-state period!
- ② Avoid calibrating within two hours after a meal or an insulin injection.
- ③ Do not calibrate when a rapid up/down arrow appears next to the glucose value.
- ④ If you calibrate in unsteady-state period, the Ref-BG will be considered invalid. Please do it later.

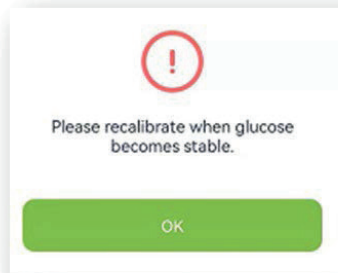- ⑤ If you forget to calibrate on time, please do it before next meal.
 - ⑥ Readings can be different, especially when the glucose fluctuates rapidly
- When “Meter reading–Receiver reading / meter reads *100 %” > 50 %, wash your hands and take a 2nd fingerstick.
 - If the value of 2nd fingerstick is similar to the first, add a calibration before next meal.


 **Warning**

If inaccurate reference blood glucose is entered, it may lead to incorrect blood glucose results.



6. Event Inputs

The user should maintain a full activity log by entering all exercise, meals, medicine, and insulin, during the entire sensor wearing period. It helps to identify the effects of events on your glucose change patterns.


Step 1: Tap “” user icon. Step 2: Tap Log Event.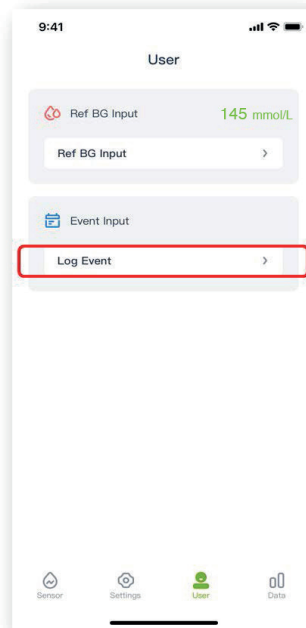

Step 3: Select the event you want to enter. And then choose OK to confirm.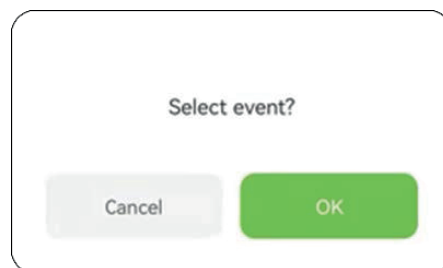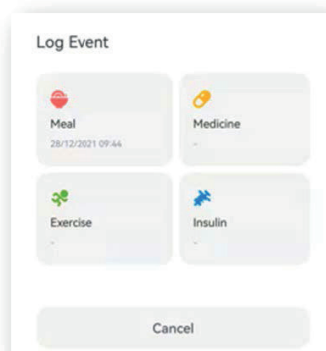

Step 4: Once an event has been entered, a colored mark on the glucose graph will be displayed, for example “Meal”.

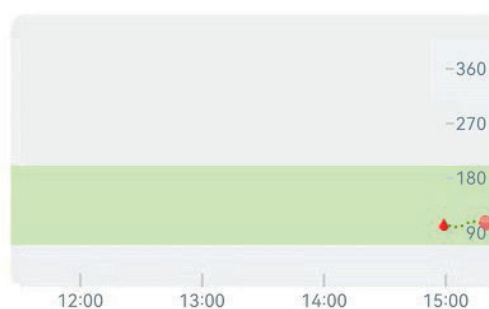7. Features of the Touch-screen display

7.1 Blood Glucose Trend Arrow

7.2 Blood Glucose Graph Color

The glucose graph is red when glucose level is below the low threshold, is orange when above the high threshold, and is green in the normal range.

7.3 Blood Glucose in Hyper/Hypo States and alerts

When you are in Hyper or Hypo status, the “🔔” will appear in the APP interface to alert you blood glucose is too low or blood glucose is too high.

Hypoglycemia predict alert:

When your blood glucose level is predicted to be lower than 55.8 mg/dL within 20 minutes, the “🔔” will appear in the APP interface with a hypoglycemia predict alert: hypoglycemia warning time for some minutes. The actual warning time is calculated based on the change trend of blood glucose, not a fixed value.

Hyperglycemia predict alert:


When your blood glucose level is predicted to be higher than 270 mg/dL within 20 minutes, the “🔔” will appear in the APP interface with a Hyperglycemia predict alert: Hyperglycemia warning time for some minutes. The actual warning time is calculated based on the change trend of blood glucose, not a fixed value.

Urgent hypoglycemia alert:

When the blood glucose value is lower than 55.8 mg/dL, the “⚠️” will appear in the APP interface together with sound and vibration to alert you are in severe hypoglycemia. The sound and vibration will not disappear automatically until the blood glucose is higher than 55.8 mg/dL, Unless you set your phone to silent mode. The text alert will not disappear automatically until the blood glucose is higher than 55.8 mg/dL.

Urgent Hyperglycemia alert:

When the blood glucose value is higher than 270 mg/dL, the “⚠️” will appear in the APP interface to alert you are in severe hyperglycemia. The sound and vibration will not disappear automatically until the blood glucose is lower than 270 mg/dL, Unless you set your phone to silent mode. The text alert will not disappear automatically until the blood glucose is lower than 270 mg/dL.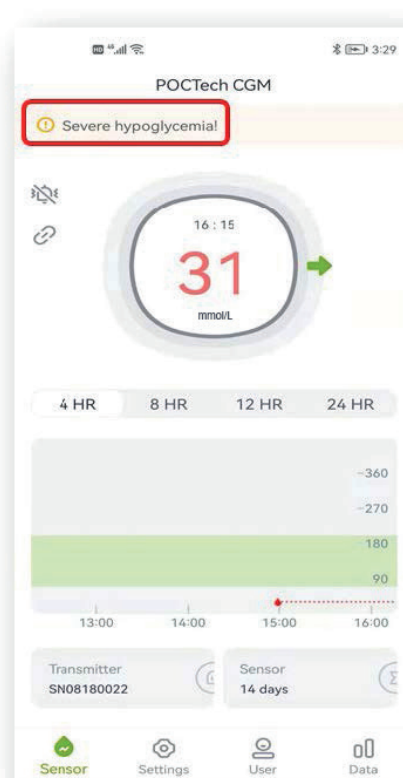

7.4 Left/Right Page Turns


Slide a finger left or right moves one page each time.

7.5 Review Glucose Point Value

Touch on any point of the glucose graph, the glucose value of this point will be shown.

8. Hypoglycemia and Hyperglycemia alert

An alert (beep or vibration) for Hypoglycemia or Hyperglycemia may be accompanied by a popup window such as shown below, you can click 'Cancel' to close it, or 'OK' to stop the prompt for 20 minutes.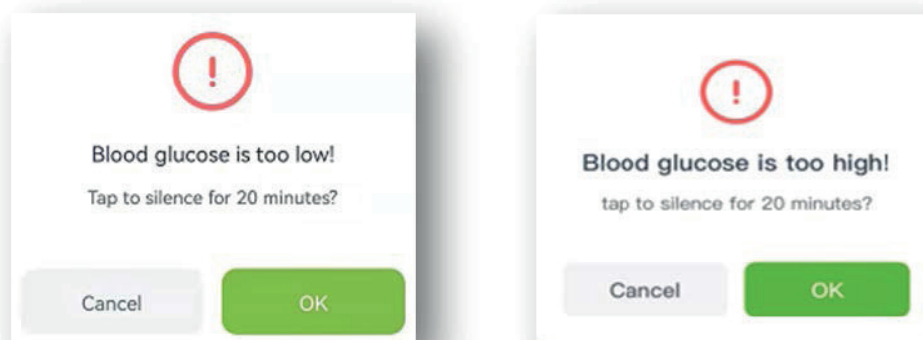

9. Ending your sensor session

9.1 Ending automatically

The system is programmed to collect data for up to 7/14 full days. It terminates the data collection automatically when a 7/14-day acquisition is complete.

9.2 Force Stop

If you want to end sensor session early, please click on the line “Click Here to turn sensor Search/On/Off >>>” and choose “Force Stop”.

A window “If Sensor is off, the test will be terminated. Please be careful.” will be displayed. Choose “OK” to end.

10. Remove the sensor together with the transmitter.

(See the Transmitter preparation of the CGMS User Manual for details.)

11. Review data

Tap the “” Menu icon, you can review data.

11.1 The sensor signal

The sensor signal is for troubleshooting when abnormality is suspected.

11.2 Review & Export Data

Review historical sensor data. The user can export the file in the form of “TXT” through the export button. The user can find the corresponding monitoring data through the given path.


11.3 24-hr Overlay

Display a 24-hr Overlay of the selected sensor data file.


24 hr Overlay

SN08180022 28/12/2021 10:47:17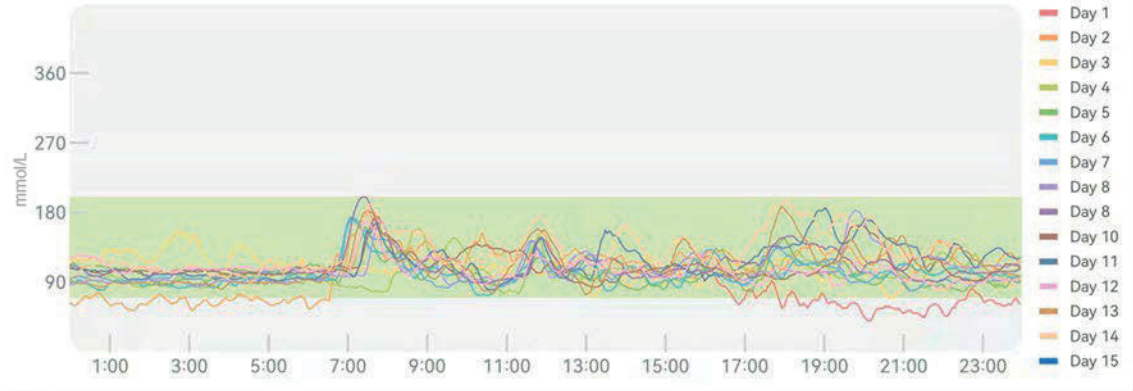

11.4 View Report

Open the Data Menu, and choose “View Report” to open a list of data files. Select a file to view a complete patient monitoring summary report.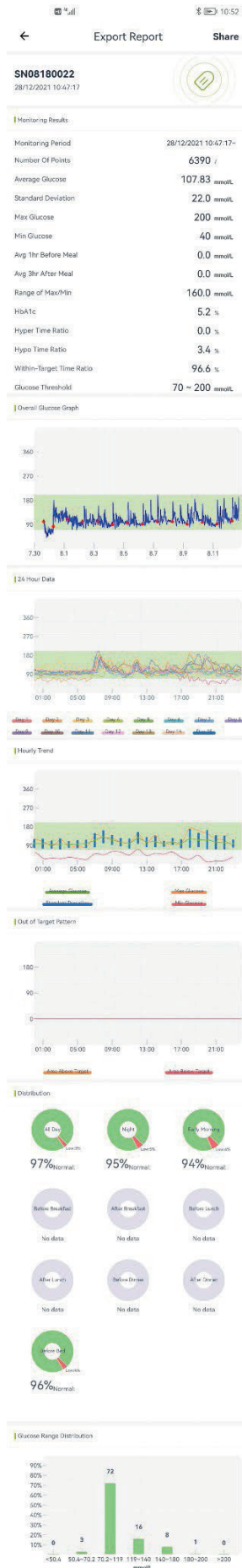


11.5 Share data

You can enter www.poctechcgm.com in the browser to access the cloud platform. It is recommended to log in through the computer, the recommended browser is Google Chrome. Firstly, you need to register your account on the cloud platform, you can use your own personal email to register and bind your device (transmitter) SN number. You can share your data after logging in to the cloud platform. You can invite your followers to view your blood glucose at the same time and share it through your account. As long as your followers log in to the cloud platform, they can view your blood glucose data in real time. During wearing period, please make sure your phone's network is turned on and it will cost some network flow, then your data can be uploaded to the cloud platform. Share isn't available on the receiver.

12. Settings

Tap "Settings" to open the settings page.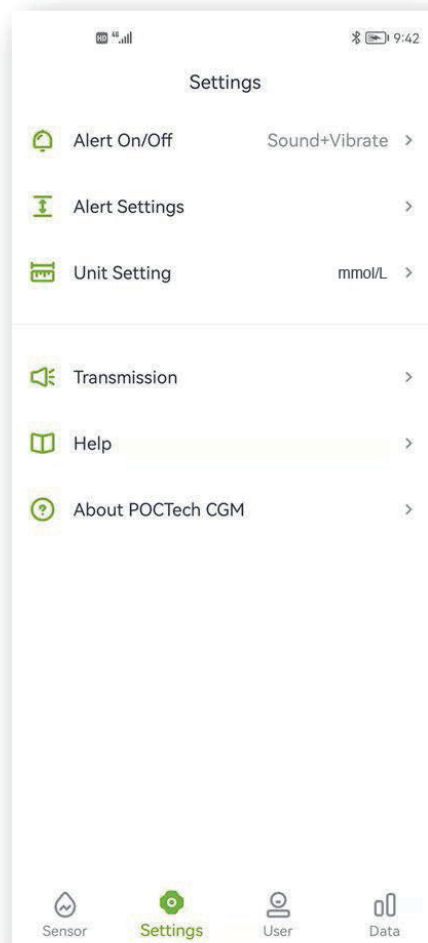


12.1 Customizing app alerts

When you get a CGM alert, such as hypoglycemia, abnormal signal, your smartphone sounds and vibrates. Customize alerts or silence them according to your needs.


Note:

If the vibration motor and/or speaker on your phone is not functioning properly you may miss alerts. Please be careful to adjust this feature or follow your doctor's instructions. Once you silence the alert, you might miss a severe low or high glucose event.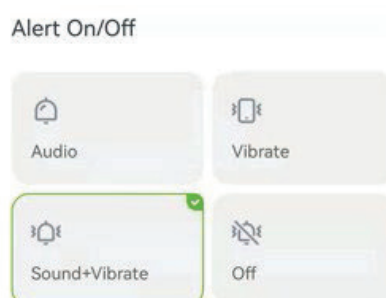

12.2 Setting high/low alerts

High alert default is set at 11.1mmol/L(200mg/dL) and low alert default is set at 3.9mmol/L(70mg/dL). Tap “Setting” to open next page to adjust high and low alerts values according to your conditions or the doctor’s advice.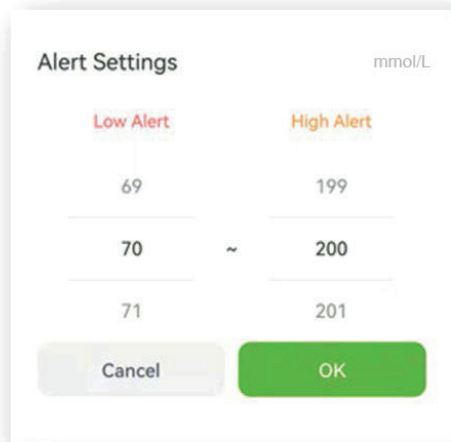


Note

To receive Alerts, you must:

- Verify app hasn't been shut down.
- Turn on Bluetooth.
- Turn off “Do Not Disturb” feature on your phone

12.3 Close the alert of audio and vibrator

You can turn off the alert of audio and vibrator if you want to do so. Note that when the alert is turned off, you may miss alert for Hypoglycemia and Hyperglycemia. The Popup Windows remain active even if you have turned off the audio and vibrator.

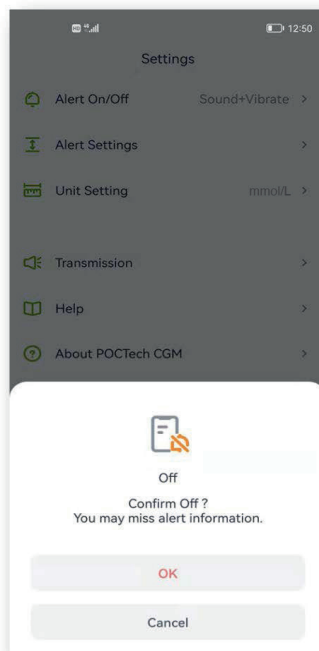13. Maintenance

13.1 Make sure that your smart phone Bluetooth is on.

Your transmitter talks to your POCtech CGM APP with Bluetooth wireless communication. Avoid using power saving modes that could turn off Bluetooth.

13.2 Manage the app to stay active

The app must always be running in the background and may drain your smart phone battery. Turn off as many apps as possible to keep your POCtech CGM APP running in the background. POCtech CGM APP and transmitter are designed to retrieve all data even when the APP becomes disconnected from time to time. Each time you touch the screen to view the APP, it will re-connect and retrieve all data that have been missed during the off-time.

13.3 Carry your smart phone at all times

It is recommended that you carry your smart phone at all times. The alert function will not be available when the transmitter is not linked. Once re-connect, it will retrieve all data that have been missed.

13.4 Never clean up the POCtech CGM APP cache

Never clean up the POCtech app cache during the monitoring period or a major error will occur which might enforce you to re-start the sensor and all stored data will be cleared.

13.5 Update software version

Since the hardware of mobile phones will be updated quickly, the POCtech APP can be used continuously for at least 2 years after it's been released. It is recommended to update the latest version in time.

13.6 Troubleshooting

No.	Icon display	Other display	Reason	Measure
1			Disconnect between app and transmitter	Click  to reconnecting or wait for reconnecting
2		Sensor signal error!	Current error: lw<4 lw>300	1. Wait for recovery. 2. Please change the sensor if could not recovery more than 15 minutes.
3		Transmitter battery low!	Transmitter battery low	Change the battery.
4		Transmitter failure. Please end the session.	Bad contact with transmitter battery	Contact customer service
5		Please turn on Bluetooth.	System Bluetooth is turned off	Open Bluetooth
6		Please recalibrate when glucose becomes stable.	Trend instability	Wait for trend stabilization and enter the parameters again
7		Connect failed, try again.	Connect fail	Connect again or restart Bluetooth and reconnect
8		Abnormal data	When the current is abnormal, try to input the reference.	1. Wait for recovery. 2. Please change the sense if could not recovery more than 15 minutes.
9		Not enough valid data to generate this report!	View historical data and reports when the sensor is just bound	Report will be generated after the first BG be input.
10		Signature error! Please download the genuine app!	APP was tampered with and not downloaded and installed from official channels	Download through the google play or official website.
11		Can't run on <i>root</i> mobile phone or simulator!	Mobile phones are <i>root</i> or try <i>root</i>	Install on a non- <i>root</i> phone

Warning

All stored data will be deleted once uninstall the POCTech CGM APP.

14. Performance

14.1 Software Performance

	Items	Performance
1	Data output interval	1data / 3min
2	Calculate response time	Within 1s
3	Data integrity rate	above 95%
4	Simultaneous connection number	1

14.2 Stability and accuracy

The stability and accuracy depend on the CGMS sensor and your calibration. (See the CGM User Manual for details.)

You can get the latest version of User Manual of at www.poctechcorp.com

15. Manufacturer information

Zhejiang POCTech Co., Ltd.
No. 1633 Hongfeng Road, Building 11 & 12,
Huzhou City 313000, Zhejiang, China

Prolinx GmbH
Brehmstr. 56, 40239 Duesseldorf,
Germany

© Zhejiang POCTech Co., Ltd. All rights reserved.

Manual may change without notice. When this happens, a new issue of this manual will be released.

You can get the latest version of User Manual of POCTech CGM APP at www.poctechcorp.com

If need help, please contact Customer Service: 86-400-118-8528 or local distributor, just like if need paper user manual as free.

Version: V20211229